Easy Power

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***The most advanced technologies for face, décolleté and body
treatments in a single device, in synergy with each other to
maximize the performance!***

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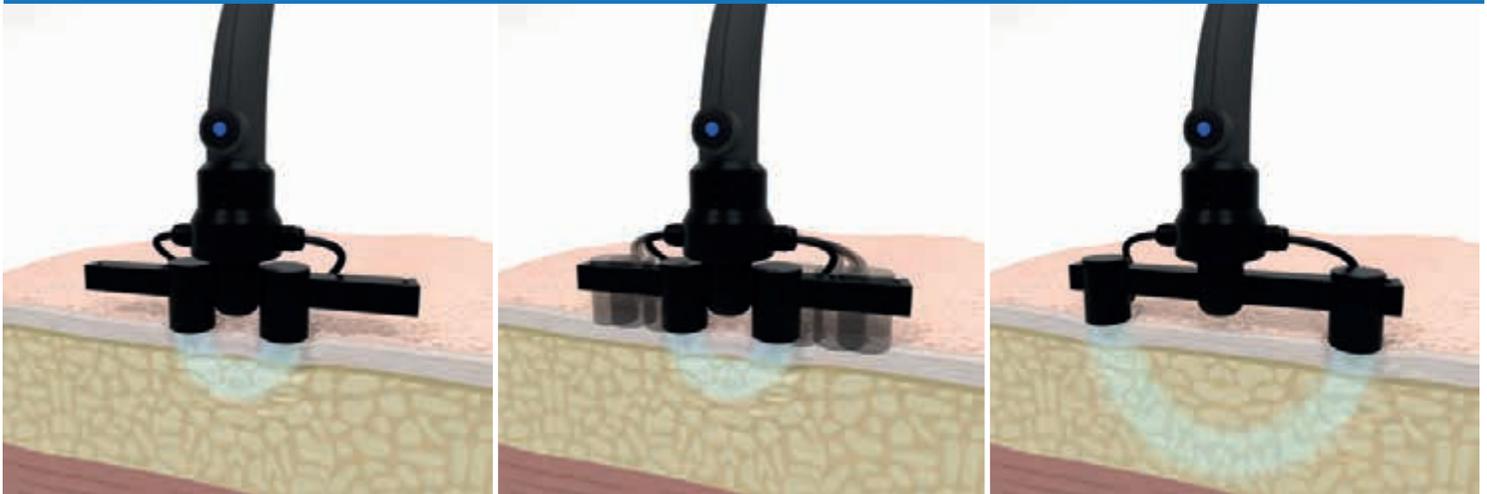
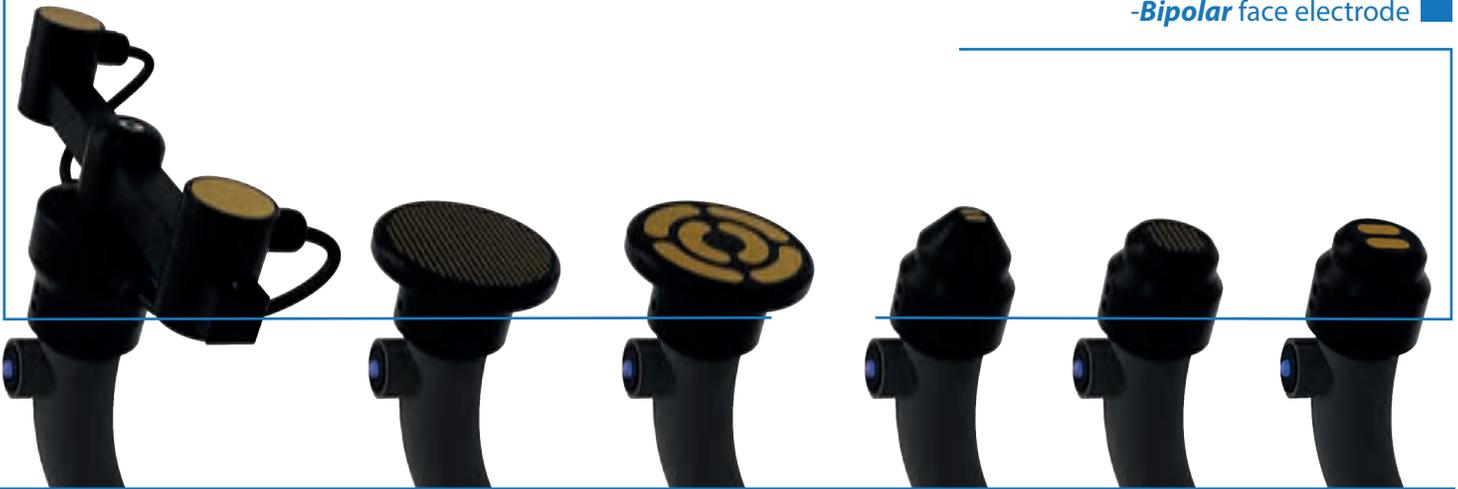
A part of the treatments is performed with a handpiece characterized by the possibility of mounting different types of heads, with different geometric characteristics and functions

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*The head with **variable geometry** for body treatments allows, by increasing or reducing the distance between the electrodes, to focus the energy delivered respectively deeper or more superficially to work exactly where you want to concentrate the treatment action.*

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• Hyperthermic radiofrequency - Bipolar FACE head + bipolar WRINKLES head

The technique allows an immediate "lifting" effect of the skin thanks to the reorganization of the connective fibers; this rapid effect will be followed by a more lasting one of neofibrillogenesis, with permanent production of new connective tissue. This allows a remodeling of the shapes, a recovery of the skin tone, the reduction of sagging folds, an overall rehydration.

• Pulsed radiofrequency - Bipolar resistive

The radiofrequency emission occurs in a pulsed way, allowing you to work even superficially and in delicate areas or in sensitive parts.

• Matrix RF - Resistive hyperthermic radiofrequency with emission points matrix

The particular "column" emission of energy is exploited to work especially at the epidermal level, concentrating the action on superficial imperfections. Micro-roughness, pigment spots, skin impurities, dilated pores, post-acne scars benefit from the treatment.

• Electropulsation

It helps the absorption of antioxidant, lightening, anti-acne, anti-aging, moisturizing substances, synergizing the action of the various types of radiofrequency chosen.

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Body treatments

• Hyperthermic radiofrequency - Resistive hexapolar

The hexapolar handpiece allows to broaden and intensify the field of action for the areas of the body. The rise in dermal temperature is exploited in excess skin adiposity and in cellulite with an increase in the adipose panniculus. The firming effects remain, thanks to the collagen and elastic neofibrillogenesis, for which application on sagging and hypotonic areas and on cellulite* with loss of tone is indicated. The improvement of microcirculation and the oxygenating,*

detoxifying and draining action have excellent indications in cellulite with stasis, edema and water retention.*

• Pulsed radiofrequency - Resistive hexapolar

In this case, the hexapolar handpiece works more superficially and is preferred for the treatment of the so-called cellulite in the early stages or in the presence of water retention and orange peel. Stretch marks with skin that has lost firmness are also an elective indication.*

• Matrix RF - Hyperthermal resistive emission dot matrix radio frequency

The particular "column" emission of energy is used to work especially at the epidermal level, concentrating the action on superficial blemishes. The blemishes of stretch marks, pigmentation spots and skin impurities find fundamental benefit from the treatment.

• Electropulsation

Especially in the body, where the thickness of the skin and the depth of the fat make aesthetic interventions difficult, it is important to combine the effect of targeted active ingredients that synergize the action of other physical energies. raining, firming, lipolytic, anti-cellulite cosmetics, etc. are indicated.*

Easy Power

DEMS – Diffuse Electro Myo Stimulation

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DEMS – Diffuse Electro Myo Stimulation

Method that uses electrical impulses of various frequency, intensity and duration to obtain a targeted contraction on certain muscle bundles, but also widespread on all minor, voluntary and involuntary muscle bundles present in the treated area. To obtain a correct and pleasant muscle movement for the treated subject, the waveform is biphasic compensated.

DEMS is applied to the muscles with an innovative system of electrically conductive bands which allow an ideal diffusion of stimulation in the treated area and allow the treatment to be concentrated on any areas with greater need. *It spread the stimulation over multiple muscle bundles and make the physical activity generated more physiological. By varying the frequency, last of the pulse and the pause, the treatment is aimed more at aerobic and therefore slimming rather than anaerobic and therefore toning rather than microcirculatory activities.*

This technique allows you to perform a contraction without tiring (therefore for all ages) and make work both voluntary and involuntary fibers.

A DEMS phase carried out after a HEM Radio wave phase benefits from the endogenous heating generated in the area and allows to use high intensity electrical stimulation without discomfort.

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DES – Draining Electro Stimulation

*Technology essentially aimed at **venous and lymphatic drainage, extremely suitable for blemishes due to excessive stagnation of localized liquids**. It takes advantage of the electrical impulses to perform a peristaltic massage through the sequential activation of the stimulating channels, which create a significant increase in lymphatic migration towards the lymphatic drains.*

Essential treatment also to drain the body after phases aimed at slimming, toning and multifactorial treatment of cellulite.*

It is applied with an innovative system of soaked electroconductive bands.

The treatment consists of two phases:

- *Phase A: characterized by electromagnetic stimulation capable of increasing peripheral circulation capillarization phase.*
- *Phase B: characterized by a peristaltic massage performed through the sequential activation of the stimulating channels.*