SLIM UP TOP CENTER®

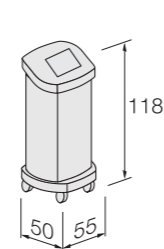
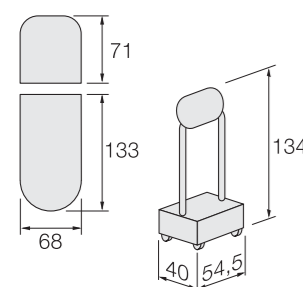
SLIM UP TOP CENTER® 2.0

SAUNA
ITALIA®

SLIM UP® DRAIN

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
*Unit for drainage and pressomassage.
3 sections: lower limbs, upper limbs, head and face.
Controlled by the TOP CARD. It may also be used manually.*

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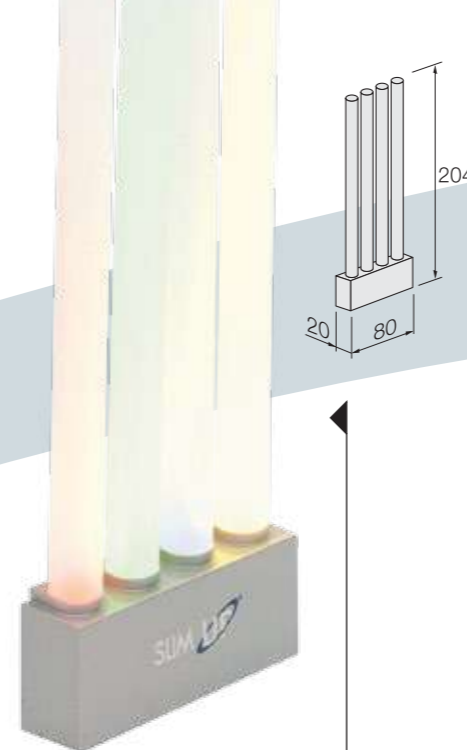*Unit for body treatment based on successive steps of heating and cooling
4 programs controlled by the TOP CARD.
It can be also manually set.*

SLIM UP® LIGHTER

Luci e colori in ambiente.
Lights and colours in your room.

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*It acquires data, stores them automatically, analyses them and indicates which treatment to perform.
It generates the TOP CARD.
It gets you in touch with the dietician.*

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*The beauty parlour uses the method and the tools of the Slim UP TOP CENTER® 2.0 for the well-being and care of the body (slimming, toning, shaping, excess weight treatment, multi-factor treatment of cellulite), together with technologies for face treatment (firming, draining etc.)
Personnel will have to be substantially trained both on technical and on sale aspects.*

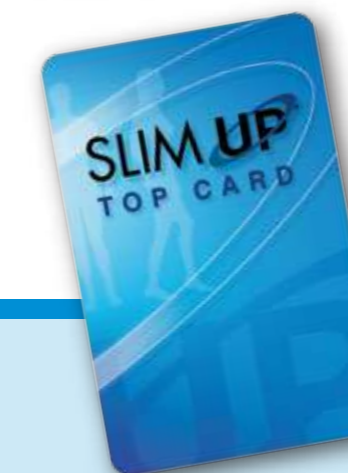
*Our doctors and instructors will hold the necessary training courses.
Our techniques will complement the practical training in the Slim UP TOP CENTER® 2.0 sections.*

*The Slim UP TOP CENTER® 2.0 is identified by a SIGN and the treatment cabins by a PLATE.
Posters, counter display cards and folders will be available.*

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

*It identifies the customer, stores the treatment cycle, selects and activates the programs on the single machines, detects the number and the type of past and future treatments.
Data uploading by transponder.*

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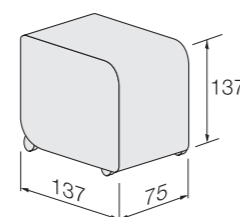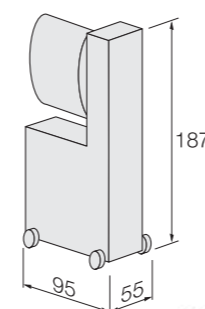
*Unit for infrared treatment with electro-stimulation and ultrasounds.
7 programmes controlled by the TOP CARD.
It may also be used manually.*

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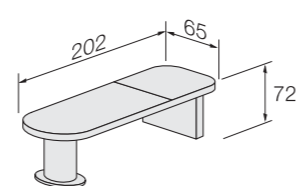*Unit for treatment by heat, steam and aromatherapy.
5 programmes controlled by the TOP CARD. It may also be used manually.
It can be replaced by Slim UP BigSteam, a unit with the same features.*

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Infrared section

- *Fat reducing, lipolytic, slimming effect, i.e. effective against overweight and obesity, due to an increasing in local metabolism. This process takes place thanks to the deep penetration of the heat which is induced through infrared rays better than other methods*
- *Lymphatic drainage effect, thanks to the faster speed of lymphatic return*
- *Cellulite blemishes reducing effect, thanks to the improved shallow micro-circulation, leading to a better local oxygenation, and to the increased lymphatic drainage and fat metabolism*
- *Better absorption of cosmetic products due to an increased micro-circulation and to the reduction of the skin electrical barrier*
- *Beneficial effect on skin trophism, i.e. on the nutrition condition of the skin and on its repair mechanisms, thanks to a better oxygenation, an improved drainage and detoxification and the stimulation of an increased reproduction of cells*
- *Relaxing and antidepressant effect thanks to the boosted production of neurohumours*
- *Analgesic and pain-relieving effect due to a better drainage, an improved oxygenation and muscle relaxation*
- *Muscle relaxation and soothing effect with boosted electro-stimulation effects*

Ultrasound section

- *Great increase of the infrared lipolytic effect: this further heating of tissues is due to the "cavitation" effect against the liquid micro-bubbles and to the "permeabilising" effect on the fat cells membrane*
- *Fibrolitic effect for the reduction of advanced cellulite imperfections and accumulation of localised fibroadiposity*
- *Capillarizing, oxygenating and detoxifying action*
- *Analgesic (pain relief) and myorelaxant action*
- *Advanced absorption of cosmetic products through the skin*
- *Fibroblasts activation, important in tissue regeneration and skin tone*

Electrostimulation section

- *Firming or toning of muscles thanks to either involuntary or induced or passive excito-motory activity*
- *Shaping of the body thanks to the readjustment of proportions*
- *Fat reducing effect due to the direct lipolytic action of electrical current*
- *Skin and under-skin firming, i.e. effect on the loss of skin and under-skin tone due to stimulation of even the thinnest muscular fasciae which cannot normally be voluntarily activated*
- *Venous draining effect due to pumping action on muscular contraction*
- *Analgesic and pain-killing effect*

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Heat

- *Lipolytic or fat reducing effect: a beneficial effect on excess of fat and obesity, it stimulates local metabolism thanks to the infrared heating, which penetrates deeper in the body than it does with other methods*
- *Beneficial effect on cellulite due to an increasing of circulation, capillarisation, oxygenation and local nutrition, an increasing of detoxification, vascular gymnastics and lipolytic effect*
- *Better penetration of cosmetic products due to the increased micro-circulation and to the reduction of the skin electrical barrier*
- *Detoxifying or detoxicant effect due to increased perspiration and the elimination of drugs, toxins, pollutants etc.*
- *Stimulating effect on the immune system and on antiviral defenses*
- *Analgesic and pain-killing effect*
- *Muscle relaxing and soothing effect*
- *Relaxing effect*

Vaporization and aroma

- *Better absorption of aromas conveyed by steam, through sweat glands which are activated at their most and through sebaceous glands which are freed from sebum*
- *Vessel toning, reducing and firming effect due to the nature of aromatic active principles*

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ThermoBalance technology

- *Alternate and sequential application of heating and cooling on deep dermal and hypodermal level*
- *Reconstruction of a regular microcirculatory system, with optimization of tissue nutrition and removal of metabolic waste and toxins*
- *Pro-collagen effect, with restoration of the tissue architecture and elasticity recovery*
- *Control of localized fat, with the increase of the tissue metabolic activity and shapes modelling*

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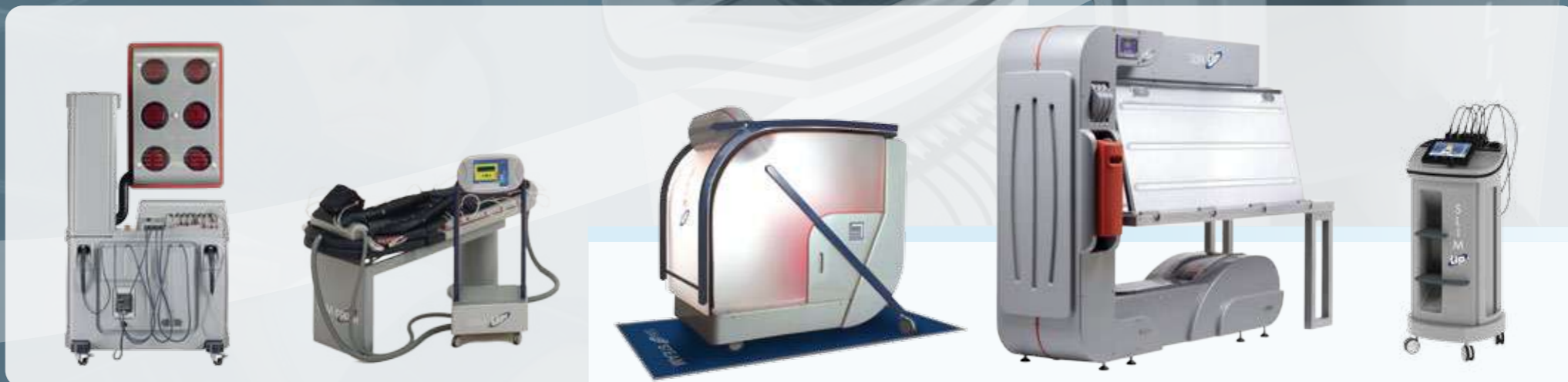
Drainage - Pressomassage

- *Venous or anti-oedema effect. i.e. an action against circulatory stasis and venous insufficiency, thus an action against swelling, heaviness of limbs, varices or varicosities*
- *Lymphatic drainage effect, which combats oedema, stagnation, stasis or heaviness*
- *Beneficial effect on cellulite blemishes, as it fights lymph and blood stasis, which is the first mover of cellulite, and thanks to its draining action*

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Slim UP[®] STATION 2.0 (patented) is a device that helps you to take care of your customers step by step, from welcoming her/him in your Center to conceiving a custom-made program which exploits all your Slim UP TOP CENTER[®] 2.0 equipment, taking care of the customer's needs and of her/his morphological characteristics, everything to achieve her/his utmost satisfaction.

The custom-made program generated by Slim UP[®] STATION 2.0 takes into account the physiological and technical motivation that have led to the development of the other units: Slim UP[®] STATION 2.0 settles the best treatment cycle for your customer and gives you the opportunity to store it onto the Slim UP[®] TOP CARD, also keeping in memory all the data of the first customer's check-up as of the following ones, creating in that way your customer's archive.

Slim UP[®] STATION is really user-friendly and easy to use through its latest generation tablet and its dedicated App. The device electronically performs all the measurement: height, body circumferences and the height of their measurement, weight and impedance. It analyzes these data and it gives you the subject's body composition, distinguishing between fat mass, intra-cellular and extra-cellular water. Everything goes automatically and it follows a preset wizard, avoiding all possible mistakes.

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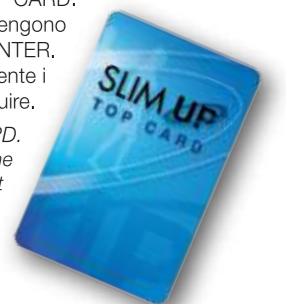
How does the App works:
- New Customer registration and Customer Archive consultation
- Setting area for the Beauty Center data and the treatment amount
- Brain Connection, a way to directly connect to Sauna Italia
- Slim UP[®] TOP CARD, to manage your customer's personal cards, watching the progress in their cycle, and to manually register BONUS CARD

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Slim UP[®] STATION 2.0 generates the TOP CARD. The treatments of the cycle are registered on the personal customer's CARD: reading the CARD, each unit of your TOP CENTER will automatically detect and apply the right treatment. Reading a CARD during the cycle, you will know which treatment have been made and which ones are still to complete.

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
General technical specifications

Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	400 W
Electrical safety Class I	
Applied accessories type BF	
Continuous operating device	

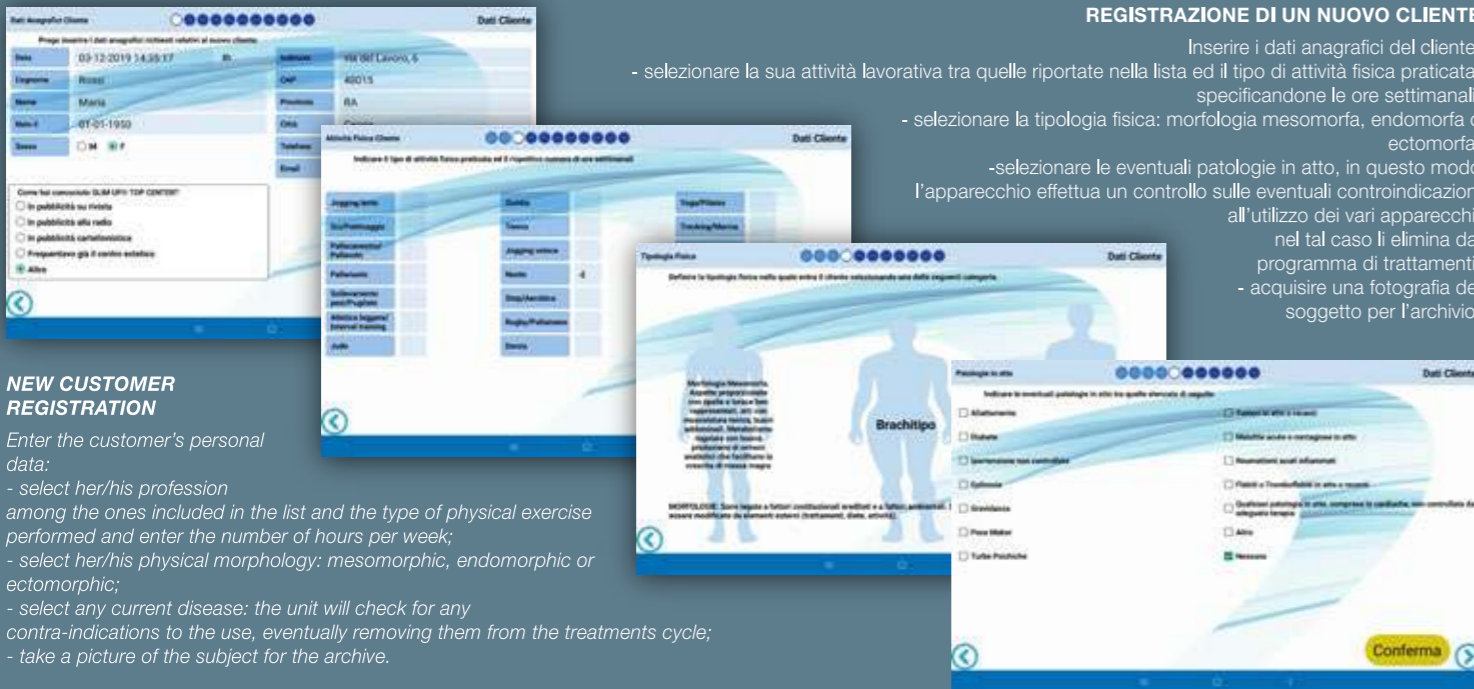NEW CUSTOMER REGISTRATION

Enter the customer's personal data:
 - select her/his profession among the ones included in the list and the type of physical exercise performed and enter the number of hours per week;
 - select her/his physical morphology: mesomorphic, endomorphic or ectomorphic;
 - select any current disease: the unit will check for any contra-indications to the use, eventually removing them from the treatments cycle;
 - take a picture of the subject for the archive.

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CHOOSING THE BEST WORKING PROGRAM

- **Slim UP your body**, to set a treatment cycle designed upon the customer's detailed measurement;
 - 4 preset programs, each one with its specifics
 - **beautify**, to improve the tone and the firmness of skin
 - **be-well**, for whole body optimal well-being
 - **age-defy**, to fight the signs of aging
 - **be-active**, to increase muscle tone or recover after an intense workout

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CUSTOMER'S MEASURING CYCLE

- **HEIGHT**: it's taken lowering the bar until it touches the customer's head: the unit will detect the data electronically;
 - **CIRCUMFERENCES**: shoulders, chest, waist, tummy hips and thighs, using the tape included in the meter: the unit will electronically detect the circumference and the height of the measurement;
 - **WEIGHT and IMPEDANCE**: electronically detected by the sliding scale and the metal-plates for hand and foot;
 - **BLEMISHES**: selecting out of the list any imperfection of the customer's body and its location.

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BODY INDEXES

The unit analyses the customer's data and it provides descriptions and visual informations about **ideal weight, fat and fat-free mass, hydration and silhouette comparison** with the ideal values.

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RESULTING CYCLE

The unit calculate the best treatment cycle for the customer's situation and shows it through a grid of sessions, with the best programs sequence; you can also watch the right placement of plates or electrodes for the treatments that need their applications. The cycle also shows you the intermediate and final controls of SlimUP STATION 2.0 and the right moment to carry out them.

SlimUP STATION 2.0 gives some general **EATING HINTS** (in case of necessity, the unit suggests the customer to follow a diet) and some **EXERCISES HINTS** to improve the tone where the customer needs, with pictures to better understand the suggested movements.

SlimUP STATION 2.0 defines the **TOTAL AMOUNT** of the cycle, depending on the costs of each single treatment you set up; there is also the possibility to choose a reduced cycle, with less sessions and a lower amount.

In the **PRINT** section, you can easily save, print or e-mail to your customer all the measurement taken, the analysis results, the amount of the treatment cycle, the eating and exercises hints.

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CUSTOMER'S ARCHIVE.

Area dedicated to each customer's data storage: anagraphic data, set of measurement taken, both the first one and the following check-ups, body indexes and resulting cycle of treatments. Through the TOP CARD you can directly access to the customer's sheet and you can see the progress of the treatments cycle.

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
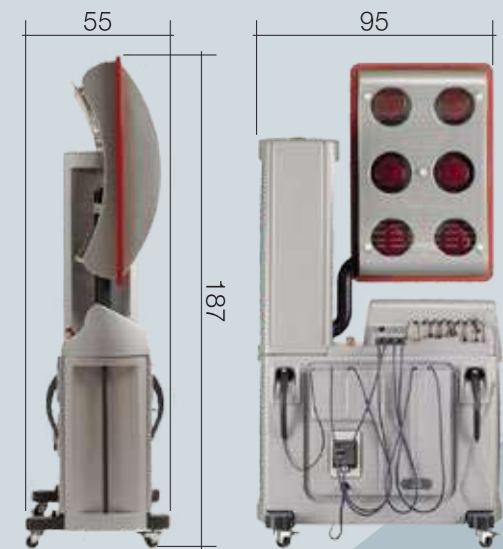
Slim UP® ULTRA offers a complete range of treatments which employ synergetic energies for globally treating unsightly body imperfections. Its absolutely unique strength lies in a careful and detailed examination of energy associations and their ability to deal with each cosmetic problem from many directions: in this way, its power is boosted by these concerted effects. The integration of a widely laboratory tested infrared-electro stimulation treatment with strong ultrasound wave sources represents a decisive innovation in the cosmetic industry, giving the professional beautician an advanced, complex yet easy-to-use tool.

But, Slim UP® ULTRA also represents an important advance concerning rejuvenating and firming face and body, through two hand pieces which open up new and fascinating horizons in cosmetic treatment.

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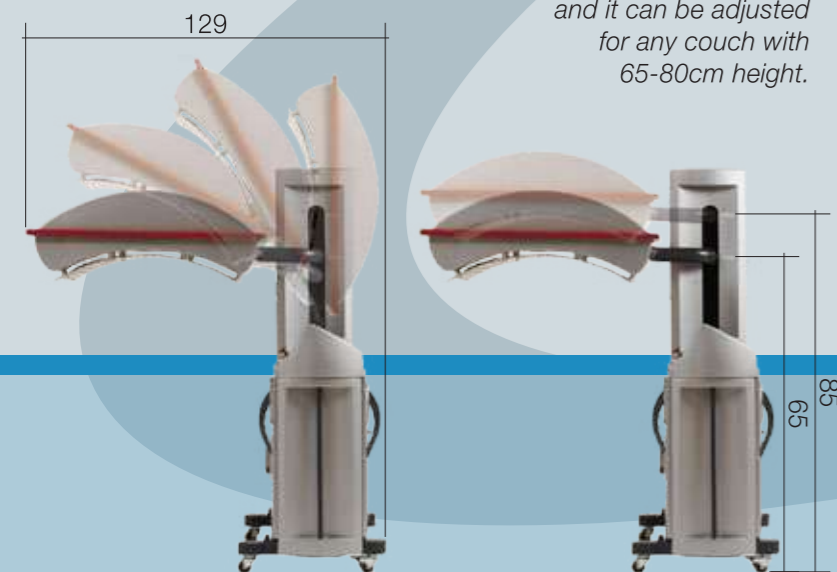SLIM UP® ULTRA takes up very little room, only half a square meter. The IR appliance is included in the device depth when not working and it can be adjusted for any couch with 65-80cm height.

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Letto SLIM UP TOP CARD
SLIM UP TOP CARD reader

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Equipped with transponder uploading system for the Top Cards in which the treatments recorded by the Slim UP® STATION 2.0 are stored; after having been performed, treatments are automatically taken off the Top Card.

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A sophisticated easy-to-use control unit, managed by microprocessors, controls all the Slim UP® ULTRA functions.

There are 7 resident programs:

- **SLIM:** aimed at slimming in presence of cellulite;
- **TONE:** to obtain improved muscle and skin tone;
- **MODEL:** to promote both a toning and a slimming effect;
- **LIPOCELL:** aimed at capillarization, improving micro-circulation and drainage;
- **MASS:** for targeted muscular mass volume increases;
- **FIBROCELL:** to actively intervene on imperfections of old and persistent cellulite;
- **MEGASLIM:** the most powerful lipolytic and fibrolytic treatment, indicated in cases of particularly accentuated adiposity.

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20 HIGH POWER BODY ULTRASOUND SOURCES

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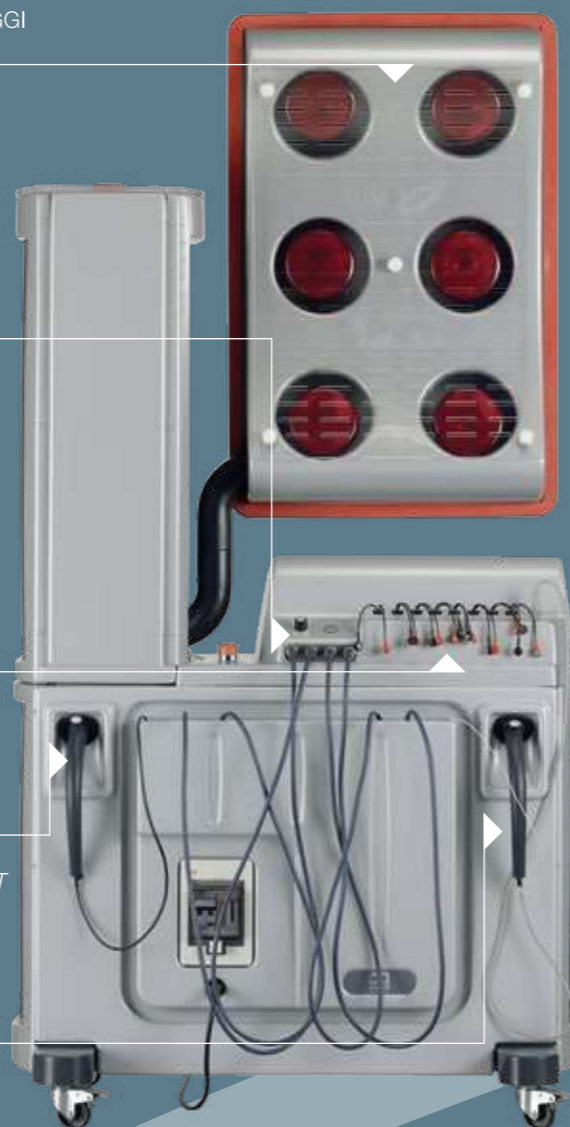
8 SOURCES FOR BODY ELECTROSTIMULATION

MANIPOLO VISO PER ULTRASUONI

FACE HAND PIECE FOR ULTRASOUND TREATMENT

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WHITE LIGHT AND ELECTROSTIMULATION HAND PIECE

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The 16 adhesive electrodes of Slim UP® ULTRA complete the series of global treatments for a more beautiful body. Modulated, bidirectional square wave current maintains and improves tone even in the most difficult areas; the use of specific frequencies and programs makes it possible to work on large, deep muscle groups for shaping the silhouette, as well as on the very thin superficial and subcutaneous muscle groups, for improving the loose skin tone. The analgesic and myorelaxing effect of the infrared rays and ultrasound waves make the Slim UP® ULTRA current very gentle and imperceptible, eliminating any and all discomfort during treatment. The association of several actions makes it possible to concretely deal with fat and cellulite, by improving skin tone, harmoniously shaping the silhouette and treating the main body imperfections.

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Slim UP® ULTRA also allows you very targeted treatments through its 2 hand-pieces. The US hand-piece for face and body activates fibroblast and it favours the diffusion of active substances for anti-age face treatment, but also in the treatment of particular areas of the body, like the inner thigh, arms, belly and breast areas. The EMS face hand-piece combines an athermic high intensity white light with electro stimulation, for toning and biostimulating trophic cutaneous effects. This face gymnastics together with the energy of this luminous band have an action of rejuvenating, anti-oxidising, smoothing, elasticizing and skin lightening, with deep muscle firming and toning effects.

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General technical specifications

Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	900 W
Available power for lamps:	600 W (for 6 lamps, 100 W each)
Electrical safety Class I	Applied accessories type BF
Continuous operating device	
Sectioning and cutout by switch high-sensitivity residual current circuit breaker	

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The heart of Slim UP® ULTRA is based on 6 sources of infrared rays with wave lengths selected from those with the power to penetrate the skin barrier and to act at a depth most useful to the beautician, i.e. the depth at which cellulite imperfections and fat cells accumulate. Thanks to the infrared rays, which raise the temperature in these areas, the metabolism of the fatty tissue is consistently increased for prolonged periods. This achieves a reducing action on fat accumulations and sculpts the silhouette.

The beneficial effects of the infrared rays are numerous: faster lymphatic drainage; improved tissue oxygenation and nutrition; regenerating, healing and anti-age effect on the skin; improved penetration of cosmetic products; muscle relaxing and pain relief action as well as a marked improvement in mood. These actions extend the rays field of action to the multifactorial treatment of cellulite, in all of its various forms, the toning of the skin and stimulating superficial and deep circulation.

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CONTROL UNIT, sophisticated but user-friendly, microprocessor-controlled, it manages all of Slim UP® STEAM functions.

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CASTOR WHEELS with brake, for an easy movement

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Slim UP® STEAM is a patented beauty therapy unit combining the thermal effect of infrared rays with the beneficial effects of steam and aromatherapy; it features five programmes alternating and combining these functions for obtaining a vessel toning, firming, reducing and slimming effect according to needs; this is an essential unit for the multi-factor treatment of cellulite.

Slim UP® STEAM helps your customers in 4 ways:

- A) a draining effect, in case of water retention, oedema, cellulite with lymph-oedemas, and excess weight due to the increase of the body liquid component*
- B) a lipolytic effect in case of excess weight*
- C) a beneficial effect in presence of any kind of cellulite, due to a better local vascularization, to the improved drainage, to the elimination of toxins and of waste products and to lipolysis*
- D) skin firming effect, due to the deep penetration of essential oils.*

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Independent by the water system thanks to its filling WATER TANK

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3 reservoirs for managing 3 different aromas

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5 automatic working programs which alternate and combine the operation of infrared rays, steam and aromatherapy (with the opportunity to select 3 different aromas) and 3 functions for the manual selection of IR, steam and aromas.

Only 3 buttons, along with a complete and immediate response of the back-lit alphanumerical display, make all functions easily selectable.

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Slim UP® STEAM uses the transponder reader for the Top Cards recorded by the Slim UP® STATION 2.0; once performed, the treatments are taken off the Card.

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General technical specifications

Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	3000 W
Available power for lamps:	900 W (for 9 lamps, 100 W each)
Power of steam generator:	2000 W
Electrical safety Class I	Continuous operating device
Sectioning and cutout by switch high-sensitivity residual current circuit breaker	

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Slim UP® Big Steam is the natural alternative of Slim UP® STEAM in the composition of your TOP CENTER: it is a device that aims to the same results, giving to your customer an even more comfortable condition thanks to its integrated couch, to totally relax during the treatment.

Using Slim UP® Big Steam too, the beneficial effects reached by the infrared thermal function together with the steam and the aromatherapy give a whole toning, firming and slimming result. Like Slim UP® STEAM, this is an essential device for the multifactorial treatment of cellulite and its imperfections.

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Displaying aroma stock: three LED signals dedicated to the aroma tanks shows when they are getting empty.

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Displaying water stock: a LED signal shows when the water tank is close to emptying.

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5 automatic working programs, exactly like the Slim UP® STEAM ones.

Only 3 control buttons and a complete and immediate response of the back-lit alphanumeric display.

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
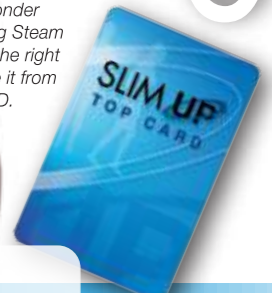
Thanks to its transponder system, Slim UP® Big Steam automatically starts the right treatment and delete it from the customer's CARD.

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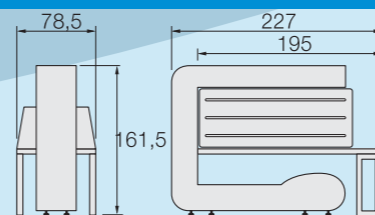General technical specifications

Input voltage: 230 V~
Main frequency: 50/60 Hz
Power consumption: 3100 W
Available power for lamps: 1100 W (for 11 lamps, 100 W each)
Power of steam generator: 2000 W
Electrical safety Class I Continuous operating device
Sectioning and cutout by switch high-sensitivity residual current circuit breaker

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Slim UP® DRAIN it's a unit for total body drainage and presso-massage: it is designed for three different sections, the head and face, the upper limbs and the lower limbs.

A sophisticated yet user friendly electronic unit, controlled by a microprocessor, manages all the Slim UP® DRAIN functions.

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Displaying parameters and single programs

Slim UP® DRAIN features a wide graphical back-lit display: it shows all the programs and every single working sequence; it focus the body areas under compression and the pressure values, both the setup and the actual applied ones (which are also graphically represented), and it shows the pause times between sectors and cycles.

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Setting treatment parameters

Slim UP® DRAIN features only 4 buttons, through them you will immediately be operating with the machine; both the software and the graphical interface have been designed to make Slim UP® DRAIN truly user-friendly.

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SLIM UP® TOP CARD

Lettura a transponder per le TOP CARD registrate dalla Slim UP® STATION 2.0: una volta effettuati, i trattamenti vengono scalati dalla Card.
Transponder reader for the Top Cards recorded by the Slim UP® STATION 2.0; once performed, the treatments are taken off the Card.

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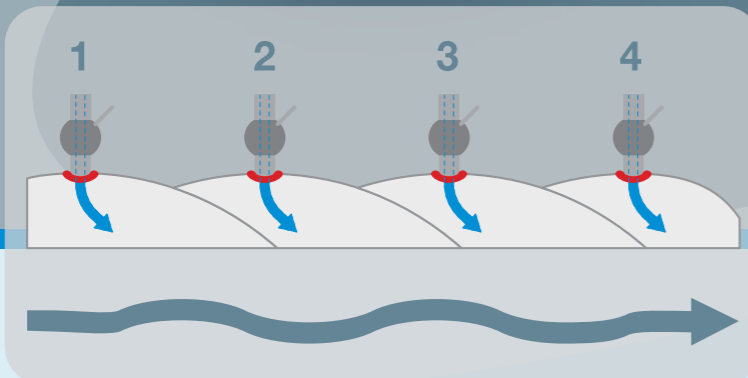
THE 3 WORKING SECTION OF SLIM UP® DRAIN

A - Head-face section

Helmet with a built-in headrest for sequential treatment of the face (chin, cheeks, forehead, front and back of neck. Front and back of neck with Sea Wave System).

The headrest has 6 outlets for a correspondent number of sectors in the helmet, as well as a system for immediate connection to the machine (RAPID-AIR).

B - Upper limbs section

2 arm-pieces with 7 Sea Wave sectors for the sequential treatment of upper limbs and an integrated palm-piece for the hand in order to exercise pressure solely on the palm and on the back of hand with no crushing at the sides.

The particular anatomy of the arm-piece, which is adjustable through a velcro fastening, additionally to the palm piece, allows you to treat the limb uninterruptedly from the hand to the upper shoulder. This section is housed in the handy (patented) hangable container, fit for immediate use: it connects to the machine with a quick coupling system, capable of recognising the corset element which it is dedicated to (RAPID AIR).

C - Section for lower limbs, hips, groin

2 leg-pieces, 2 thigh-pieces, 2 groin compressors, 1 corset element for treating hips and buttocks. Each leg-piece consists of 4 Sea Wave sectors and of an arch support made by a sole of self-moulding material, a rigid shaped sole and an air bladder, designed for exercising pressure under the sole of the foot and never at its sides.

The pressure is applied from the sole of the foot all along the limb with no interruption. The thigh-piece makes it possible to adjust the extension of the leg-piece and to adapt it to the different heights of the limbs. The anatomically designed groin compressor allows a perfect lymphatic drainage of the groin area.

The corset for hips and buttocks is made of one piece only and consists of 2 Sea Wave sectors.

The adjustment of the corset elements through a velcro fastening allows you to extend them to very different sized anatomical parts.

This section is housed in the handy (patented) hangable container, fit for immediate use: it connects to the machine with a quick coupling system, capable of recognising the corset element which it is dedicated to (RAPID AIR).

Sea Wave is a system made up of overlapping segments: each of these is sequentially inflated before the previous one is totally deflated. This enables a selective and progressive pressure without interruptions.

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B - CONTAINER (patented)
it houses the upper limbs section (hangable)

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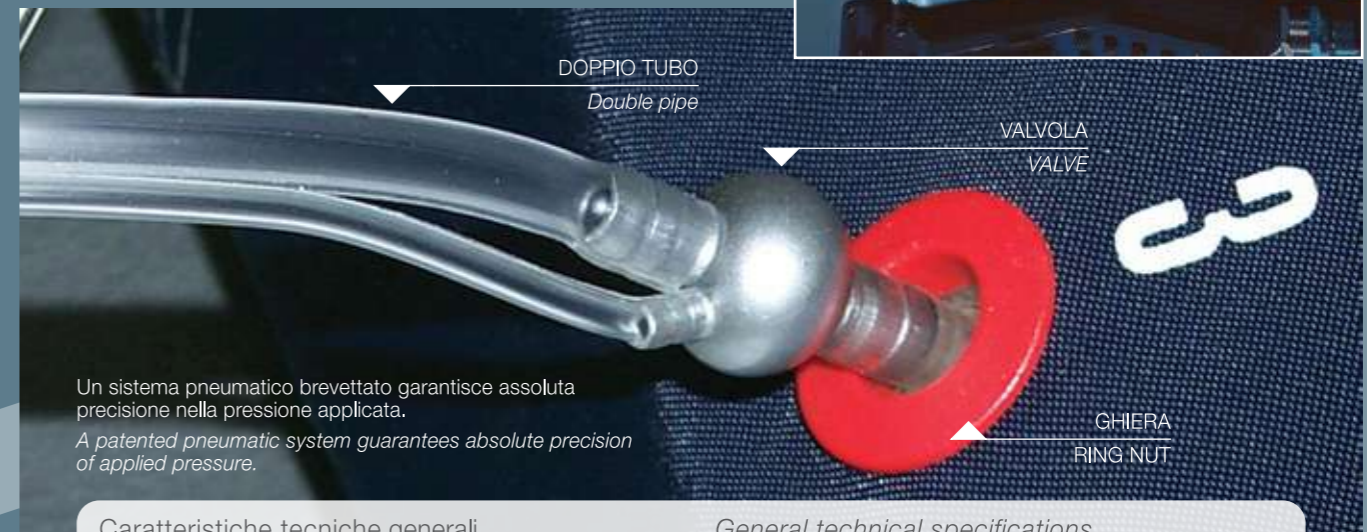
C - CONTAINER (patented)
it houses the lower limbs section (hangable)

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A patented pneumatic system guarantees absolute precision of applied pressure.

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Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	300 W
Electrical safety Class I	
Applied accessories type BF	
Continuous operating device	

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A totally new approach to imperfections that we are facing everyday, the so-called cellulite, loss of tone and localized fat excess.

Slim UP® ThermoBalance reconstructs an optimal anatomical and metabolic situation of the tissue deep layers, by recreating a correct microcirculatory environment with adequate arteriovenous exchanges, by stimulating the production of vital new collagen and reducing the excess of localized fat.

Six dedicated and exclusive plates are the heart of Thermobalance system, able to combine for the first time in a targeted manner the radiofrequency heating to Peltier cell cooling system.

This innovation passes through a unique project of pregelled single-client adhesive electrodes, an innovative heat dissipation system and an exclusive method of monitoring, control and independent modulation of all treatment process.

A targeted scientific experimentation confirmed widespread normalization and improvement of the tissue architecture as a whole.

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An innovative technology.

The beauty treatments with this technology lead to acting on different levels connected and interacting:

- Reconstruction of a regular microcirculatory system, with optimization of tissue nutrition and removal of metabolic waste and toxins;
- Pro-collagen effect, with restoration of the tissue architecture and elasticity recovery;
- Control of localized fat, with increase of the tissue metabolic activity and shapes modelling;

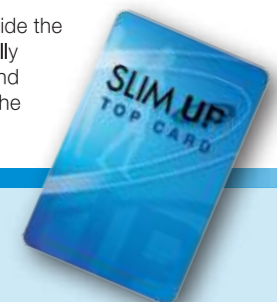
This technology is based on **the alternate and sequential application of heating and cooling on deep dermal and hypodermal level.**

The heating is generated by bipolar resistive radiofrequency, applied by special pregelled single-client electrodes, specifically studied for this application. The cooling is generated by exclusive Peltier cell thermoelectrical systems.

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The transponder inside the consolle automatically recognizes, starts and finally deletes from the CARD the right treatment.

SLIM UP® ThermoBalance

Trattamenti Slim UP® Thermobalance.

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Slim UP® Thermobalance treatments.

The treatments with this technology provide for various alternate heating and cooling phases of the treated area without the need of constant operator presence; in fact, after positioning the plates in the area to be treated, it is enough to adjust the RF intensity, then, the device controls independently the emitted intensity and the cooling according to preset parameters.

- **CELL:** for the treatment of the so-called cellulite imperfections, it works with quick alternance of hot and cold; this reactivates the arterio-venous microcirculation, the venous-lymphatic drainage and pushes to a reorganisation of the cutaneous and subcutaneous structure. The treatment ends with a heating phase to eliminate toxins and metabolites.

- **LIPO:** program aimed to localized fat excess. It mainly focuses on local metabolic activity increase. The treatment starts with a dilated cooling phase, that quickly increases the metabolism, and then restore a more correct blood supply.

- **SKIN TONE:** program aimed to the treatments of skin tone loss. The targeted and deep radiofrequency allows a visible "shrink" effect and stimulates the new production of collagen fibres. The cooling phase thickens the interstitium and works on the tissue architecture and correct vascular bed.

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ThermoBalance treatment plates (patented)

The 6 plates are engineered exclusively for the Thermobalance method. The release, the control and the modulation of both hypothermia and hyperthermia require an advanced and smart system.

The cooling effect is generated by Peltier cell thermoelectrical systems and requires a special and relevant system of heat dissipation. That leads to the application of exclusive materials and an innovative design: this makes the plates as a unique and unmistakable object.

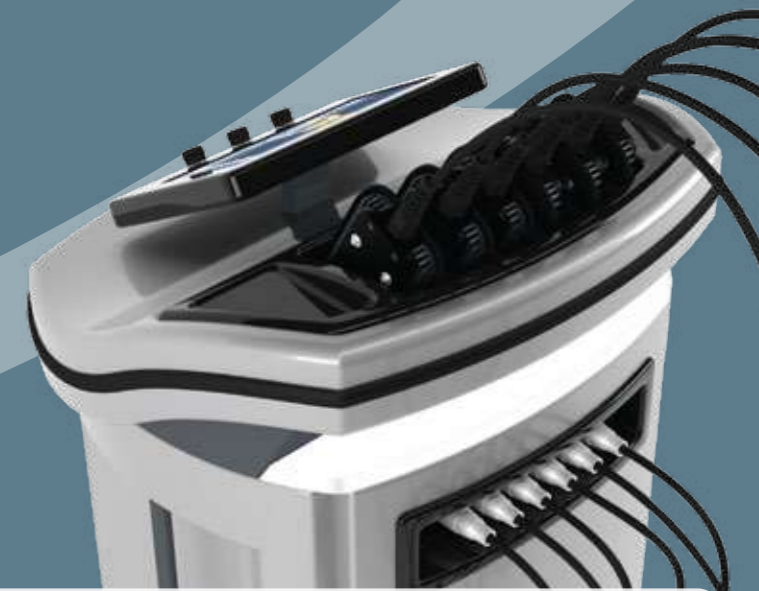
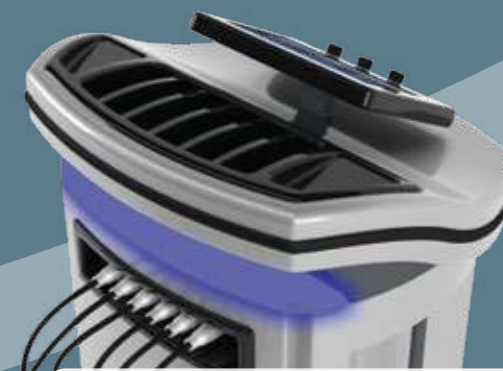
The radiofrequency hyperthermia passes through special pregelled single-client electrodes, specifically studied for this application. The electrodes have a specific shape that optimizes the treatment performances and allows a continuous monitoring of the temperature exactly in the treated area; they are made of materials specifically tested for modulable radiofrequency.

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Active treatment phase signal

Slim UP® Thermobalance has a LED signal, on the back of the console, that shows to the user the active phase of the treatment ongoing: it lights red during the heating phases and blue during the cooling ones. When the unit is in stand-by, it emits a white evanescent light.

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Input voltage:	100-240 V~
Main frequency:	50/60 Hz
Power consumption:	300 W
Electrical safety Class I	
Applied accessories type BF	
Continuous operating device	

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There are pink, the colour of joy and relax, and blue, which stands for calm and confidence. There are yellow, a symbol for wisdom and heat, and green, an expression of harmony and of spirit. There are also sounds. Sounds of far-away and near worlds, intense notes that come deep from the soul. While eyes perceive, the body listens. A couch, a machine, and a miracle takes place. Rather than a miracle, this is the ecstasy of the senses. Music enlivens the mind, the machine enlivens the body. Regaining wellness has never been so easy. Regaining beauty has never be so pleasant.

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Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	450 W
Available power for lamps:	400 W (for 4 lamps, 100 W each)
Electrical safety Class I	
Continuous operating device	

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This company has a Quality System certified by the TÜV Rheinland accreditation body according to UNI EN ISO 9001:2015 - UNI EN ISO 13485:2016 standards

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Certain treatments are not recommended during pregnancy, in presence of some heart diseases or psychiatric problems. Warning and contraindications are described in each appliance user manual. These manuals are also available for customer's consultation before purchasing.

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DAM S.r.l.