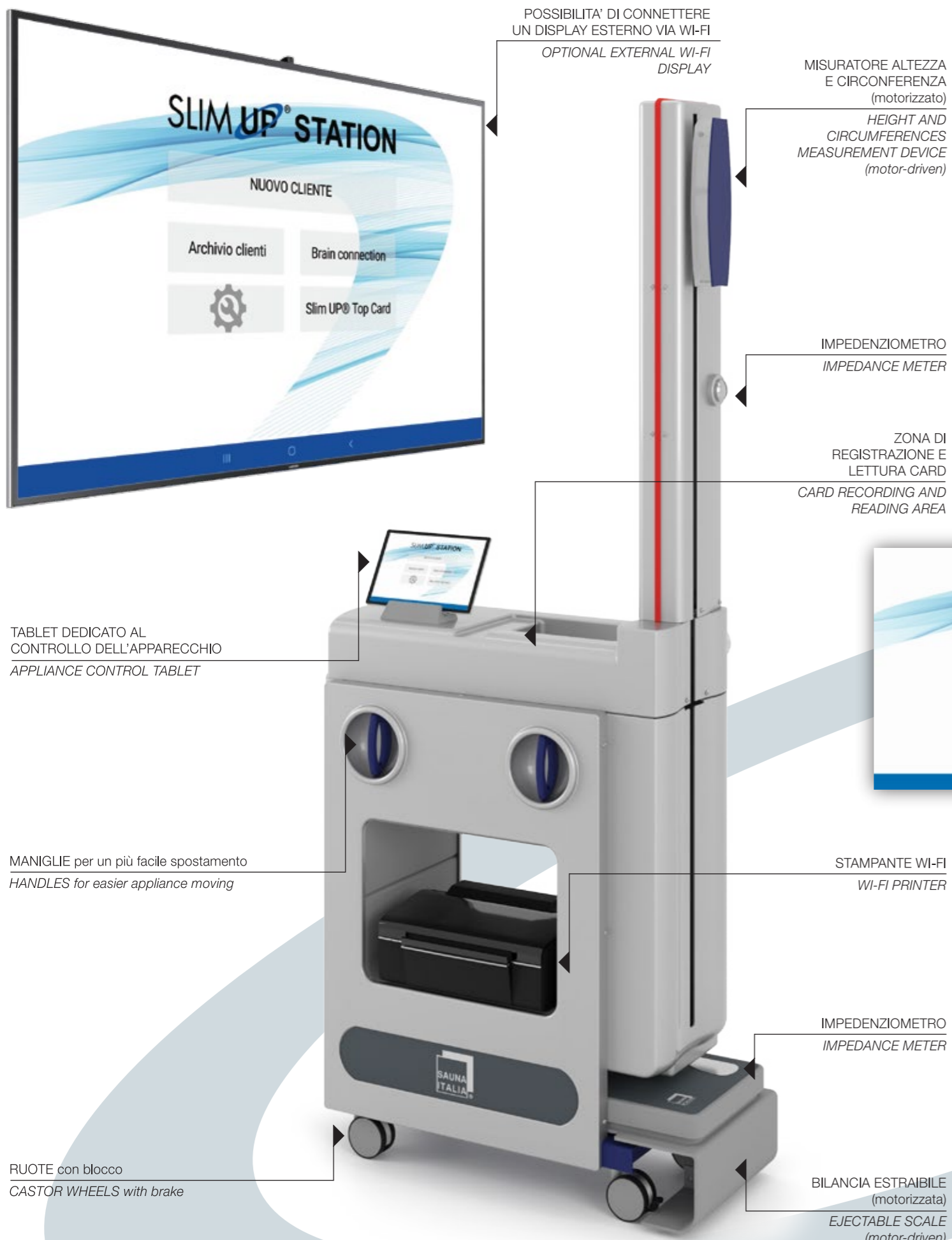
SLIM UP STATION 2.0

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Slim UP® STATION 2.0 (patented) is a device that helps you to take care of your customers step by step, from welcoming her/him in your Center to conceiving a custom-made program which exploits all your Slim UP TOP CENTER® 2.0 equipment, taking care of the customer's needs and of her/his morphological characteristics, everything to achieve her/his utmost satisfaction.

The custom-made program generated by Slim UP® STATION 2.0 takes into account the physiological and technical motivation that have led to the development of the other units: Slim UP® STATION 2.0 settles the best treatment cycle for your customer and gives you the opportunity to store it onto the Slim UP® TOP CARD, also keeping in memory all the data of the first customer's check-up as of the following ones, creating in that way your customer's archive.

Slim UP® STATION is really user-friendly and easy to use through its latest generation tablet and its dedicated App. The device electronically performs all the measurement: height, body circumferences and the height of their measurement, weight and impedance. It analyzes these data and it gives you the subject's body composition, distinguishing between fat mass, intra-cellular and extra-cellular water. Everything goes automatically and it follows a preset wizard, avoiding all possible mistakes.

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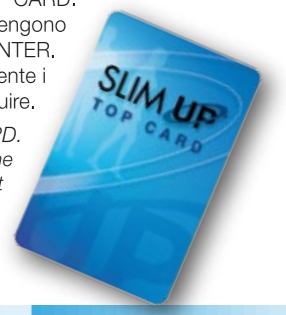
- How does the App works:
- New Customer registration and Customer Archive consultation
 - Setting area for the Beauty Center data and the treatment amount
 - Brain Connection, a way to directly connect to Sauna Italia
 - Slim UP® TOP CARD, to manage your customer's personal cards, watching the progress in their cycle, and to manually register BONUS CARD

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Slim UP® STATION 2.0 generates the TOP CARD. The treatments of the cycle are registered on the personal customer's CARD: reading the CARD, each unit of your TOP CENTER will automatically detect and apply the right treatment. Reading a CARD during the cycle, you will know which treatment have been made and which ones are still to complete.

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
General technical specifications

Input voltage:	230 V~
Main frequency:	50/60 Hz
Power consumption:	400 W
Electrical safety Class I	
Applied accessories type BF	
Continuous operating device	

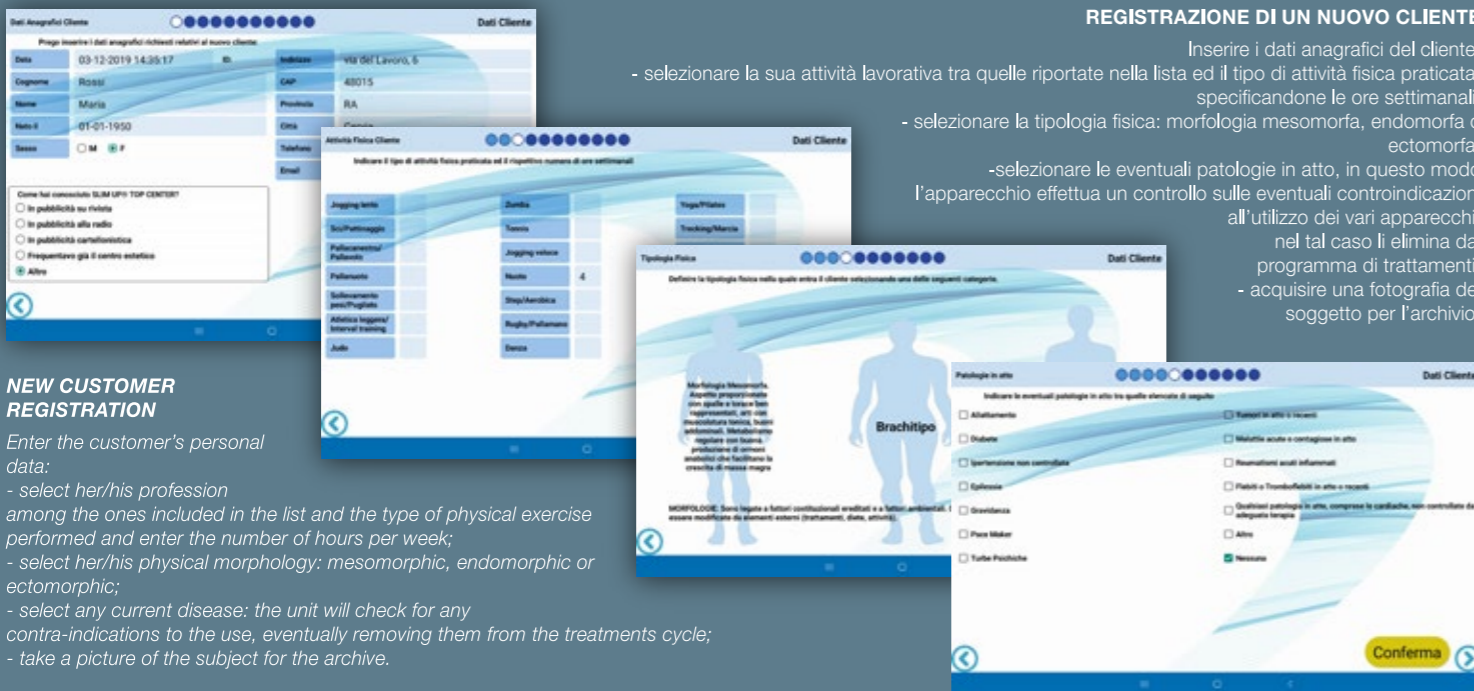NEW CUSTOMER REGISTRATION

- Enter the customer's personal data:
- select her/his profession among the ones included in the list and the type of physical exercise performed and enter the number of hours per week;
 - select her/his physical morphology: mesomorphic, endomorphic or ectomorphic;
 - select any current disease: the unit will check for any contra-indications to the use, eventually removing them from the treatments cycle;
 - take a picture of the subject for the archive.

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CHOOSING THE BEST WORKING PROGRAM

- **Slim UP your body**, to set a treatment cycle designed upon the customer's detailed measurement;
- 4 preset programs, each one with its specifics
 - **beautify**, to improve the tone and the firmness of skin
 - **be-well**, for whole body optimal well-being
 - **age-defy**, to fight the signs of aging
 - **be-active**, to increase muscle tone or recover after an intense workout

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CUSTOMER'S MEASURING CYCLE

- **HEIGHT**: it's taken lowering the bar until it touches the customer's head: the unit will detect the data electronically;
- **CIRCUMFERENCES**: shoulders, chest, waist, tummy hips and thighs, using the tape included in the meter: the unit will electronically detect the circumference and the height of the measurement;
- **WEIGHT and IMPEDANCE**: electronically detected by the sliding scale and the metal-plates for hand and foot;
- **BLEMISHES**: selecting out of the list any imperfection of the customer's body and its location.

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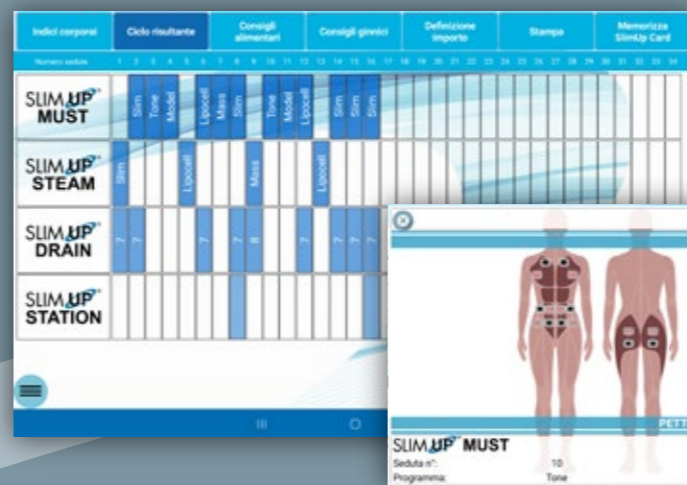
BODY INDEXES

The unit analyses the customer's data and it provides descriptions and visual informations about **ideal weight, fat and fat-free mass, hydration and silhouette comparison** with the ideal values.

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RESULTING CYCLE

The unit calculate the best treatment cycle for the customer's situation and shows it through a grid of sessions, with the best programs sequence; you can also watch the right placement of plates or electrodes for the treatments that need their applications. The cycle also shows you the intermediate and final controls of SlimUP STATION 2.0 and the right moment to carry out them.

SlimUP STATION 2.0 gives some general **EATING HINTS** (in case of necessity, the unit suggests the customer to follow a diet) and some **EXERCISES HINTS** to improve the tone where the customer needs, with pictures to better understand the suggested movements.

SlimUP STATION 2.0 defines the **TOTAL AMOUNT** of the cycle, depending on the costs of each single treatment you set up; there is also the possibility to choose a reduced cycle, with less sessions and a lower amount.

In the **PRINT** section, you can easily save, print or e-mail to your customer all the measurement taken, the analysis results, the amount of the treatment cycle, the eating and exercises hints.

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CUSTOMER'S ARCHIVE.

Area dedicated to each customer's data storage: anagraphic data, set of measurement taken, both the first one and the following check-ups, body indexes and resulting cycle of treatments. Through the TOP CARD you can directly access to the customer's sheet and you can see the progress of the treatments cycle.

SLIMUP[®] STATION 2.0